Manufactured for:

Respironics Inc.
1001 Murry Ridge Lane
Murrysville, PA 15668 USA

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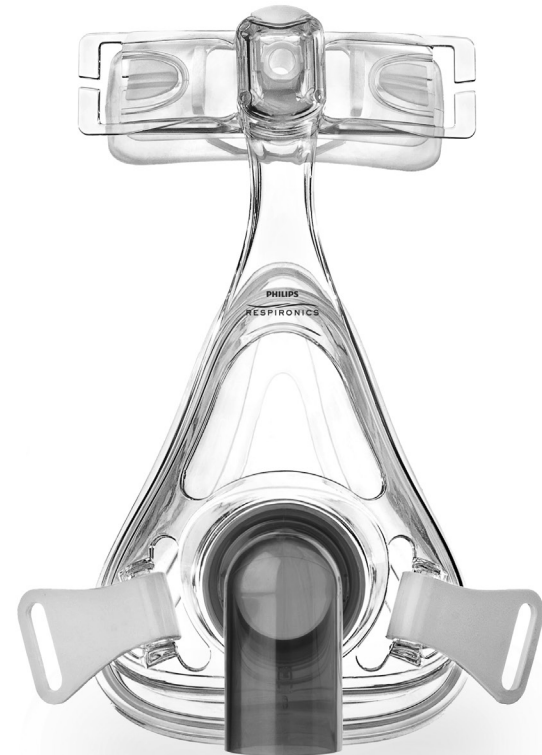
ENGLISH

Intended Use

The Amara SE and Amara Gel SE Full Face Masks are intended to provide a patient interface for application of noninvasive ventilation. The masks are to be used as an accessory to ventilators which have adequate alarms and safety systems for ventilation failure, and which are intended to administer CPAP or positive pressure ventilation for treatment of respiratory failure, respiratory insufficiency or obstructive sleep apnea. The masks are for multi-patient use in the hospital/institutional environment only. The masks are to be used on patients (>66 lbs/30 kg) who are appropriate candidates for noninvasive ventilation.

Note: An exhalation port is not built into the mask. A separate exhalation device must be used with this mask.

Note: This mask is not made with natural rubber latex or DEHP.

Symbols

Warning
or
Caution

Note

Tip

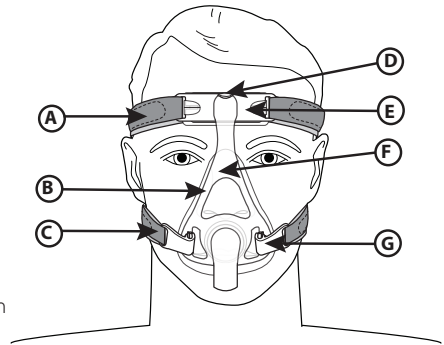
Not Made
With Natural
Rubber Latex

Warnings:

- This mask is not suitable for providing life support ventilation.
- Hand wash prior to use. Inspect the mask for damage or wear (cracking, crazing, tears, etc). Discard and replace any components as necessary.
- This mask requires a separate exhalation device.
- This mask is designed for use with CPAP or bi-level systems recommended by your health care professional or respiratory therapist. Do not wear this mask unless the CPAP or bi-level system is turned on and operating properly. Do not block or try to seal the exhalation port. Explanation of the Warning: CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation. This warning applies to most models of CPAP systems.
- This mask does not include an anti-asphyxia valve that would allow a patient to breathe if the ventilator failed.
- If oxygen is used with the device, the oxygen flow must be turned off when the device is not operating. Explanation of the Warning: When the device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the device enclosure. Oxygen accumulated in the device enclosure will create a risk of fire.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP and bi-level machines.
- At low CPAP or EPAP pressures the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
- Some users may experience skin redness, irritation, or discomfort. If this happens, discontinue use and contact your healthcare professional.

- Consult a physician if any of the following symptoms occur: Unusual chest discomfort, shortness of breath, stomach distension, belching, severe headache, blurred vision, drying of the eyes, eye pain or eye infections.
- This mask should not be used on patients who are uncooperative, obtunded, unresponsive, or unable to remove the mask.
- This mask is not recommended if the patient is taking a prescription drug that may cause vomiting.
- Consult a physician or dentist if you encounter tooth, gum, or jaw soreness. Use of a mask may aggravate an existing dental condition.
- Attaching an exhalation device requires therapy pressure level adjustment to compensate for increased leak.
- A minimum of 3cm H₂O (hPa) must be maintained when using this mask.
- The mask contains small parts which could result in a choking hazard.
- Remove the O₂ enrichment attachment when not in use.

▲ Figure 1

- A Top Headgear Straps
- B Integrated Cushion/Faceplate
- C Bottom Headgear Straps
- D Push Button Adjuster
- E Forehead Pad Support
- F Frame with Elbow
- G Headgear Clips

Contraindications

This mask may not be suitable for use on patients with the following conditions: recent eye surgery or dry eyes, hiatal hernia, excessive reflux, impaired cough reflex, impaired cardiac sphincter function; or on patients unable to remove the mask by themselves.

Before Use Read and Understand the Instructions Completely

- Hand wash the mask.
- Wash your face. Do not use moisturizer/lotion on your hands or face.
- Inspect the mask and replace it if the cushion has hardened or is torn, or if any parts are broken.
- Verify that the therapy device, i.e., ventilator, including the alarms and safety systems, has been validated prior to use.
- Verify therapy device pressure(s).

Cleaning Instructions

Hand wash the mask before first use and daily. The headgear should be hand washed weekly, or as needed. The headgear does not need to be removed for daily cleaning of the mask.

1. Hand wash mask in warm water with liquid dishwashing detergent.
2. Rinse thoroughly. Air dry completely before use. Make sure the mask is dry before use. Lay the headgear flat or line dry. Do not place the headgear into the dryer.

⚠ Caution: Do not use bleach, alcohol, cleaning solutions containing bleach or alcohol, or cleaning solutions containing conditioners or moisturizers.

⚠ Caution: Any deviation from these instructions may impact the performance of the product.

⚠ Caution: Inspect the mask for damage or wear (cracking, crazing, tears, etc). Discard and replace any components as necessary.

Institutional Disinfection

For **multi-patient use** in the hospital/institutional environment, use the **Disinfection Guide** to reprocess the mask between patients. These instructions can be obtained by visiting us online at www.philips.com/respironics or by contacting Philips Respironics Customer Service at 1-800-345-6443 (USA or Canada) or at 1-724-387-4000.

Dishwashing Instructions

In addition to hand washing, the mask may be cleaned in the dishwasher once a week.

⚠ Caution: Use a mild liquid dish washing detergent only to wash the mask.

1. Remove the headgear. Do not wash the headgear in the dishwasher.
2. Separate the cushion and the faceplate and wash in the top shelf of the dishwasher.
3. Air dry. Make sure the mask is dry before use.
4. Inspect all parts for damage or wear; replace any parts that have visibly deteriorated (cracking, crazing, tears, etc). Replace the mask if the cushion becomes hardened.

Philips Respironics System One Resistance Control

This symbol  may appear on your Philips Respironics therapy device or mask.

The System One setting is not applicable to this mask, or any masks requiring a separate/additional exhalation device.

Achieving the Right Fit

Before putting on the mask:

1. **Headgear clips** - Grip and twist away from the frame to disconnect.
2. **Headgear** - Loosen to a large setting.

Adjusting the mask:

1. Pull the mask over your head. Hold the mask against your face.
2. **Headgear clips** - Press into place to connect (Figure 2).
3. **Headgear** - Pull back the tabs to evenly adjust the top and bottom straps (Figure 3).
4. **Forehead adjuster button** - Press and hold to slide the frame toward or away from your face to adjust pressure (Figure 4). This will decrease or increase the pressure on the bridge of your nose and chin.

The mask should rest comfortably against your face. The forehead pad will touch your forehead lightly. The bottom of the mask cushion should rest just above your chin with your mouth slightly open.

Using the mask:

1. Connect the flexible tubing (included with your CPAP or bi-level device) to the elbow on the mask.
2. Lie down. Turn the therapy device on. Breathe normally. Keep your mouth slightly open.
3. Assume different sleeping positions. Move around until comfortable. If there are any excessive air leaks, make final adjustments. Some air leaking is normal.

* Comfort Tips

- The most common mistake is overtightening the headgear. The headgear should fit loose and comfortable. If your skin bulges around the mask or if you see red marks on your face, loosen the headgear.
- Adjust the bottom headgear straps to reduce leaks at the sides of your mouth.

Removing the Mask

Disconnect the headgear clips and pull over the top of your head (Figure 5).

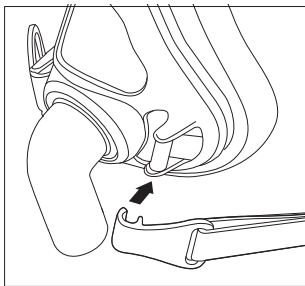
Figure 2

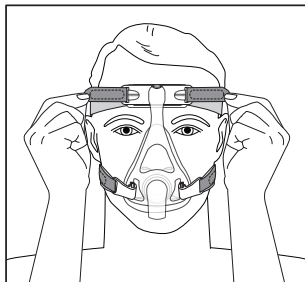
Figure 3

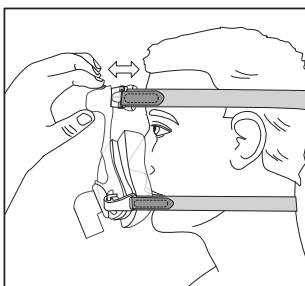
Figure 4


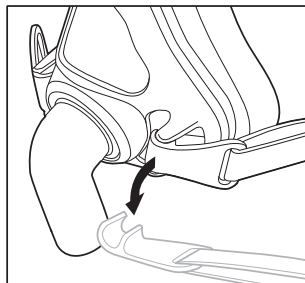
Figure 5

Disassembly of the Mask


1. **Headgear** - Remove from the forehead pad support and headgear clips.
2. **Forehead pad** - Slide off the track of the forehead pad support (Figure 6).
3. **Mask Cushion** - Grasp and pull it off the mask frame (Figure 7).

Assembly of the Mask

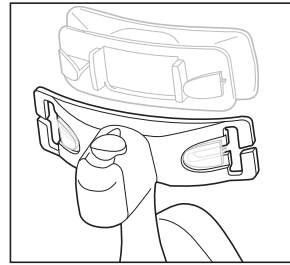
1. **Forehead pad** - Slide on the track of the forehead pad support.
2. **Mask Cushion** - Use your thumbs to press the cushion past the retaining tabs on the elbow and press into place. (Figure 8).
3. **Headgear** - Thread the top straps through the forehead pad support, and the bottom straps through the headgear clips. Push the headgear clips onto the mask frame.

▲ Figure 6

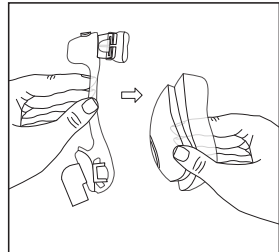▲ Figure 7

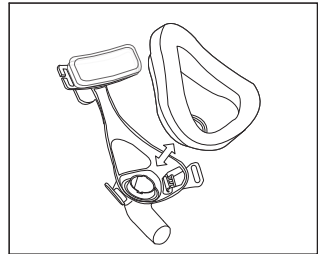
▲ Figure 8

Specifications

⚠ Warning: The technical specifications of the mask are provided for your healthcare professional to determine if it is compatible with your CPAP or bi-level therapy device. If used outside these specifications, or if used with incompatible devices, the mask may be uncomfortable, the seal of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect device function. The pressure flow curve shown below is an approximation of expected performance. Exact measurements may vary.

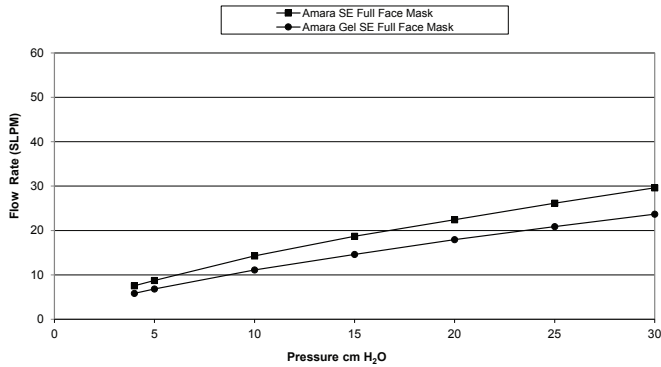Pressure Flow Curve

Resistance

Drop in Pressure at

	50 SLPM	100 SLPM
All sizes	0 cm H ₂ O	0.2 cm H ₂ O

Deadspace Volume

P	238 ml
S	245 ml
M	256 ml
L	267 ml

Sound Levels

	Amara SE Full Face Mask	Amara Gel SE Full Face Mask
A-weighted Sound Power Level	42 dBA	34 dBA
A-weighted Sound Pressure Level at 1 m	34 dBA	26 dBA

Disposal

Dispose of in accordance with local regulations.

Storage Conditions

Temperature: -4° to 140° F (-20° to +60° C)
Relative Humidity: 15% to 95% non-condensin

Limited Warranty

Respironics, Inc. warrants that its mask systems (including mask frame and cushion) (the "Product") shall be free from defects of workmanship and materials for a period of ninety (90) days from the date of purchase (the "Warranty Period").

If the Product fails under normal conditions of use during the Warranty Period and the Product is returned to Respironics within the Warranty Period, Respironics will replace the Product. This warranty is nontransferable and only applies to the original owner of the Product. The foregoing replacement remedy will be the sole remedy for breach of the foregoing warranty.

This warranty does not cover damage caused by accident, misuse, abuse, negligence, alteration, failure to use or maintain the Product under conditions of normal use and in accordance with the terms of the product literature, and other defects not related to materials or workmanship.

This warranty does not apply to any Product that may have been repaired or altered by anyone other than Respironics. Respironics disclaims all liability for economic loss, loss of profits, overhead, or indirect, consequential, special or incidental damages which may be claimed to arise from any sale or use of the Product. Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

THIS WARRANTY IS GIVEN IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IN ADDITION, ANY IMPLIED WARRANTY, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IS SPECIFICALLY DISCLAIMED. SOME JURISDICTIONS DO NOT ALLOW DISCLAIMERS OF IMPLIED WARRANTIES, SO THE ABOVE LIMITATION MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS UNDER LAWS APPLICABLE IN YOUR SPECIFIC JURISDICTION.

To exercise your rights under this limited warranty, contact your local authorized Respironics, Inc. dealer or Respironics, Inc. at 1001 Murry Ridge Lane, Murrysville, Pennsylvania 15668, USA, or Respironics Deutschland GmbH & Co KG, Gewerbestraße 17, 82211 Herrsching, Germany.

Limited Warranty for Australia and New Zealand only

Note: For Australian and New Zealand customers this warranty replaces the warranty contained above.

Respironics, Inc., a Philips Healthcare company warrants that the Products shall be free from defects of workmanship and materials and will perform in accordance with the Product specifications. 2. This warranty is valid for a period of 30 days from the date of purchase from an authorised Respironics dealer. 3. If the Product is found to contain a defect of workmanship or materials or fails to perform in accordance with the Product specifications during the applicable warranty period, Respironics will repair or replace, at its option, the defective material or part. 4. The customer is responsible for returning the product to an authorised Philips Respironics dealer; and collecting the product from the authorised Philips Respironics dealer after repair or replacement, at its own cost. Philips Respironics is responsible only for the freight cost of transporting the product between the authorised Philips Respironics dealer and Respironics. Respironics reserves the right to charge an evaluation and postage fee for any returned Product as to which no problem is found following investigation. 5. This warranty does not cover damage caused by accident, misuse, abuse, alteration, and other defects not related to materials or workmanship. 6. The warranty provided by Respironics herein is not transferrable by the Buyer in the event of any sale or transfer of Products purchased by the Buyer from an authorised Respironics dealer. 7. To the extent permitted by law, where the Buyer has the benefit of an implied guarantee under the Australian Consumer Law, but the Product is not of a kind ordinarily acquired for personal, domestic or household use or consumption Respironics' liability shall be limited, at the option of Respironics, to the replacement or repair of the Product or the supply of an equivalent Product. 8. To exercise your rights under this warranty, contact your local authorised Philips Respironics dealer. A list of all authorised dealers is available at the following link: http://www.healthcare.philips.com/au_en/homehealth/distributors_index.wpd.

Alternatively, you can make a claim under this warranty by contacting Respironics directly at: Philips Electronics Australia Limited, 65 Epping Road, North Ryde NSW 2113, Australia. Tel: 1300 766 488, Email: prcontact@philips.com. 9. The following statement is provided to a Buyer who is a "consumer" under the Australian Consumer Law: Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the good repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure. 10. The following statement is provided to a Buyer who is a "consumer" under the Consumer Guarantees Act 1993, New Zealand: Our goods come with guarantees that cannot be excluded under the Consumer Guarantees Act 1993. This guarantee applies in addition to the conditions and guarantees implied by that legislation.

AUSTRALIAN SPONSOR DETAILS:

Philips Electronics Australia Ltd.
65 Epping Road, North Ryde, NSW 2113
Australia

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©2016 Koninklijke Philips N.V. All rights reserved.