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Please read these instructions carefully before using your mask.

**RECOMMENDED USE**

The Breeze Nasal Comfort Mask is designed for nasal ventilation via the use of a barometric device delivering Continuous Positive Airway Pressure (CPAP) or bi-level pressure (BiPAP).

The Breeze Nasal Comfort Mask must be:

- used by adult patients (>30Kg) for whom a positive airway pressure treatment was prescribed.
- used by a single patient treated at home or in a health care facility.

**WARNINGS**

- Do not block or attempt to obstruct the leak ports.
- This mask can only be used with CPAP or BiPAP devices recommended by a physician or lung specialist.
- The mask cannot be used if the device has not been started up and is functioning normally.
- Explanation: CPAP and BiPAP devices must be used with specific masks equipped with leak ports enabling the continuous discharge of exhaled air. When the device is functioning normally, fresh air from the device will evacuate the exhaled air through the leak ports of the mask. However, when the device is not operational, there is not enough fresh air coming into the mask, and the exhaled air could be inhaled again. Breathing in exhaled air for longer than a few minutes can, in certain circumstances, lead to suffocation. This applies to most CPAP or BiPAP devices.*
- Stop using this mask if you experience any adverse reaction whatsoever during usage (claustrophobia, irritation, pain, conjunctivitis, etc...).
- Should this occur, consult your GP.
- Using a mask can cause pain in the gums, the teeth or the jaw or can aggravate dental problems. If this occurs, please see a dentist.
- Do not use the mask if the packaging has been opened or damaged. Replace the mask or the equipment if it is damaged or distorted.
- Clean the mask before using it for the first time.
- Do not use if the patient cannot remove the mask unaided.
- If oxygen is to be introduced, follow the instructions and safety precautions associated with the use of oxygen.
- To minimize leakage, different cup and forehead support sizes are available. Ask your health care provider.

**MAINTENANCE / CLEANING**

The mask must be cleaned every day in accordance with the following instructions:

- Separate the mask from any other accessory (harness, corrugated tube).
- Wash the different parts of the mask with warm water and a mild detergent (5 <pH < 8) such as soap or washing-up liquid.
- Rinse every part with clean water.
- Allow the equipment to dry completely on a clean cloth out of the sun before using it again.
- Wash the harness by hand once a week.

**Note:**

- Do not use bleach, alcohol or cleaning solutions containing alcohol, shampoo or strongly scented products.
- If part of the system becomes visibly deteriorated (crack, fissure, tear, etc...), the component must be replaced.

**STORAGE**

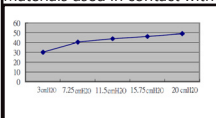
The mask must be washed and dried before being put away.

Store the mask in a clean and dry area, out of the sun.

To protect the environment, the mask must be disposed of in a waste bin. Do not discard improperly.  
Storage life : 5 years from the date of manufacture.

**TECHNICAL CHARACTERISTICS**

Treatment pressure :	3 cmH <sub>2</sub> O to 25 cmH <sub>2</sub> O
Operating temperature :	+15° C to +25° C
Storage temperature :	+15° C to +25° C
Pressure drop measurement :	at 50 l/min 0.2 cmH <sub>2</sub> O and at 100 l/min 0.85 cmH <sub>2</sub> O
Noise level :	27.5 dBA to 10cmH <sub>2</sub> O
Materials used in contact with the patient :	silicone, Oeko-Tex textile and SBR elastomer


Pressure-flow curve :

3 cmH <sub>2</sub> O :	30.1 l/Min.
7.25 cmH <sub>2</sub> O :	40.6 l/Min.
11.5 cmH <sub>2</sub> O :	44.0 l/Min.
15.75 cmH <sub>2</sub> O :	46.3 l/Min.
20 cmH <sub>2</sub> O :	49.1 l/Min.

**PUTTING THE MASK IN PLACE**

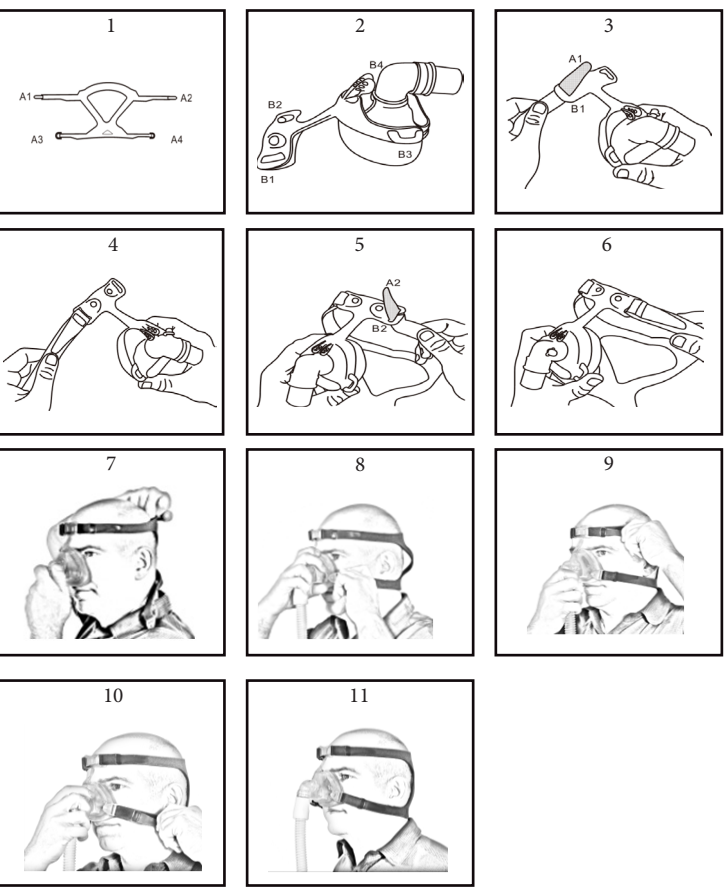Prepare the harness by detaching the Velcro at A1 and A2.  
Insert the A1 Velcro strip in B1 and close (fig. 3 and 4). Repeat for A2 and B2 (fig. 5 and 6).  
Place the mask on your head and adjust the tightness of the upper tethers of the harness (fig. 7).  
Clip the A3 tether to the B3 hook of the frame. Repeat for A4 and B4. (fig. 8).  
Adjust the tightness of the A1, A2, A3 and A4 Velcros to position the harness properly. (fig. 9 and 10)  
Make sure that the back of the harness is properly positioned, to prevent slipping.  
In case of daily use, just clip the A3 tether to the B3 hook of the frame, or A4 to B4 (fig.8), and remove it at the end of treatment.

List of components :	
1 Silicone cup	6 Plug
2 Internal frame	7 Tether clip
3 External frame	8 Adapter
4 Forehead support	9 Harness
5 Elbow	

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La mascarilla Breeze Nasal Comfort debe ser:

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Mdi Europa GmbH
HSINER Co. Ltd.