Wisp Nasal Mask
Wisp Youth Nasal Mask

PHILIPS
RESPIRONICS

Manufactured for:
Respironics Inc.
1001 Murry Ridge Lane
Murrysville, PA 15668 USA

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Instructions for use - English

Intended Use

The Wisp Nasal Mask is intended to provide an interface for application of CPAP or bi-level therapy to patients. The mask is for single patient use in the home or multi-patient use in the hospital/institutional environment. The mask is to be used on patients (>66lbs/30kg) for whom CPAP or bi-level therapy has been prescribed.

The Wisp Youth Nasal Mask is intended to provide an interface for application of CPAP or bi-level therapy to patients. The mask is for single patient use in the home or multi-patient use in the hospital/institutional environment. The mask is to be used on patients 7 years or older (>40 lbs/18 kg) for whom CPAP or bi-level therapy has been prescribed.

Notes:

- Exhalation is built into this mask so a separate exhalation device is not required.
- This mask is not made with natural rubber latex or DEHP.

Symbols

Warning or Caution

Note

Tip

Not Made with Natural Rubber Latex

System One Resistance Control


Warnings

- This mask is not suitable for providing life support ventilation.
- Hand wash prior to use. Inspect the mask for damage or wear (cracking, crazing, tears, etc). Discard and replace any components as necessary.
- Some users may experience skin redness, irritation, or discomfort. If this happens, discontinue use and contact your healthcare professional.
- Consult a physician or dentist if you encounter tooth, gum, or jaw soreness. Use of a mask may aggravate an existing dental condition.
- Consult a physician if you experience the following symptoms while using the mask or after removing it: Drying of the eyes, eye pain, eye infections, or blurred vision. Consult an ophthalmologist if symptoms persist.
- The mask material should be away from your eyes.
- Do not block or seal off the exhalation ports.
- Do not overtighten the headgear straps. Watch for signs of overtightening, such as excessive redness, sores, or bulging skin around the edges of the mask. Loosen the headgear straps to alleviate symptoms.
- This mask is designed for use with CPAP or bi-level systems recommended by your health care professional or respiratory therapist. Do not wear this mask unless the CPAP or bi-level system is turned on and operating properly. Do not block or try to seal the exhalation port. **Explanation of the Warning:** CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. This warning applies to most models of CPAP systems.
- If oxygen is used with the device, the oxygen flow must be turned off when the device is not operating. **Explanation of the Warning:** When the device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the device enclosure. Oxygen accumulated in the device enclosure will create a risk of fire.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP and bi-level machines.
- At low CPAP or EPAP pressures the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
- Attaching an exhalation device requires therapy pressure level adjustment to compensate for increased leak.

If your mask contains magnetic headgear clips, the following warnings apply:

- The headgear clips contain magnets. Contact your healthcare professional before you use this mask.
- Some medical devices may be affected by magnetic fields. The magnetic clips in this mask should be kept at least 2 in. (50 mm) away from any active medical device with special attention to implanted devices such as pacemakers, defibrillators and cochlear implants.
- Do not use in or near magnetic resonance imaging (MRI) equipment.
- Keep unassembled magnetic headgear clips out of reach of children. In case of accidental swallowing, seek medical assistance immediately.

Features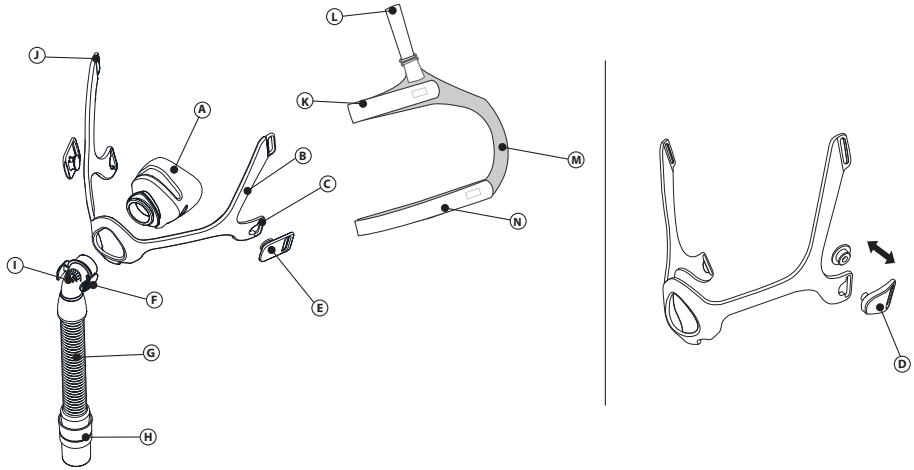

- (A) Mask Cushion
- (B) Mask Frame (clear or fabric)
- (C) Frame Bottom Slot
- (D) Magnetic Headgear Clip (optional)
- (E) Headgear Clip
- (F) Quick Release Tabs
- (G) Tubing Assembly
- (H) Swivel
- (I) Elbow with Built-in Exhalation (do not block)
- (J) Frame Top Slot Headgear
- (K) Headgear Top Strap
- (L) Headgear Crown Strap with Loop for Tube Management (optional)
- (M) Headgear Back Strap
- (N) Headgear Bottom Strap

Before Use Read and Understand the Instructions Completely.

- Hand wash the entire mask ①.
- Wash face. Do not use moisturizer/lotion on hands or face.
- Inspect the mask. Replace the cushion if hardened or torn. Replace any parts that are broken.
- If the mask has magnetic headgear clips, ensure that they are assembled properly into the frame and headgear.

Cleaning Instructions

⊖ Note: If your mask has magnetic headgear clips, leave them attached to the frame before cleaning.

Daily: Hand wash the non-fabric parts.

Weekly: Hand wash the fabric parts.

1. Wash in warm water with liquid dishwashing detergent.
2. Rinse thoroughly.
3. Air dry completely before use.

⚠ Cautions:

- Do not clean the mask with bleach, alcohol, solutions containing bleach or alcohol, or solutions containing conditioners or moisturizers.
- Inspect the mask for damage or wear (cracking, crazing, tears, etc). Discard and replace any components as necessary.
- Any deviation from these instructions will impact the performance of the product.

Dishwashing Instructions

Weekly: Additionally, the non-fabric mask parts may also be cleaned in the dishwasher.

1. Disassemble.
2. Wash the non-fabric parts in the top shelf of the dishwasher with liquid dishwashing detergent.
3. Air dry completely before use.

⚠ Caution:









Inspect the mask for damage or wear (cracking, crazing, tears, etc). Discard and replace any components as necessary.

Institutional Disinfection

⊖ Note: If your mask has magnetic headgear clips, remove them from the frame before institutional disinfection. Refer to the *Disassembly/Assembly of Magnetic Headgear Clips* section for removal.

Refer to the Disinfection Guide for Professional Users to reprocess between patients in a clinical setting. Access the latest version of the Disinfection Guide at www.healthcare.philips.com or by contacting customer service at 1-724-387-4000 or Respiration Deutschland at +49 8152 93060.

Achieving the Right Fit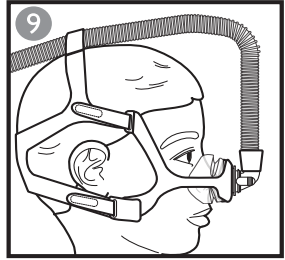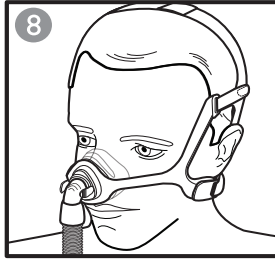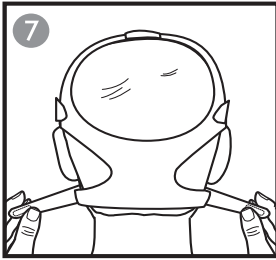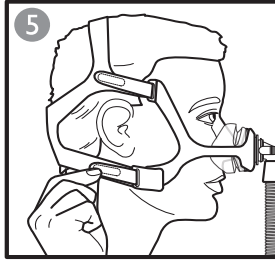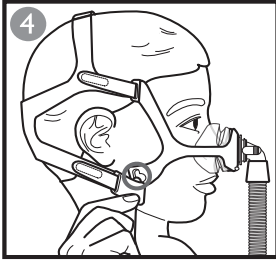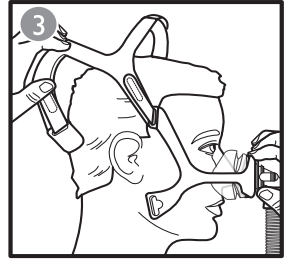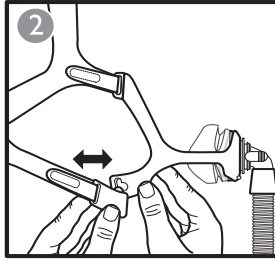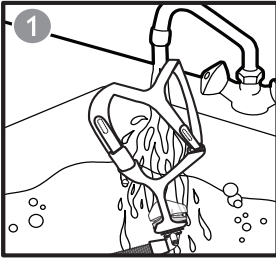

Note: Do not overtighten the headgear.

Before Putting on the Mask

1. Sizing: Select the cushion that fits the width of the nose without blocking the nostrils. The top of the cushion may sit between the bridge and tip of the nose.

Putting on the Mask

2. Headgear tab or clip: Disconnect one, or both (2). Peel the headgear tabs away from the headgear strap and pull through slot in the mask frame or if using the clip twist outward.
3. Headgear: Pull headgear on over head (3).
4. Cushion: Place the cushion over nose.
5. Headgear tab or clip: Thread tab through slot in the mask frame or press the clip against the headgear slot to reattach (4).

Adjusting the Mask

6. Headgear: Peel the headgear tabs away from the head gear straps then press tabs against the headgear straps to reattach (5).
 - a Connect the side strap to the headgear tab located at the base of the crown strap.
 - b Connect the bottom strap tab near the back of the headgear.
 7. Headgear: Adjust the side straps evenly (6).
 8. Headgear: Adjust the bottom straps evenly (7).
- The mask should rest comfortably against face (8).

Using Tube Management (Optional)

The flexible tubing may be placed over head.

- Slide the mask tubing through the headgear crown loop (9) before attaching the flexible tubing.

Using the Mask





- Connect the flexible tubing (included with the CPAP or bi-level device) to the swivel.
- Turn the therapy device on. Lie down. Breathe normally.
- Assume different sleeping positions. Move around until comfortable. If there are any excessive air leaks, make final adjustments. Some air leaking is normal.

✳ Comfort Tips

- The most common mistake is overtightening headgear. The headgear should fit loose and comfortable. If the skin bulges around the mask or if red marks appear, loosen the headgear.
- Re-seat the mask by pulling it directly away from face and gently setting it back into place.
- Adjust the side headgear straps to reduce leaks around eyes.

Removing the Mask

Disconnect a bottom headgear clip. Slide the headgear forward over head.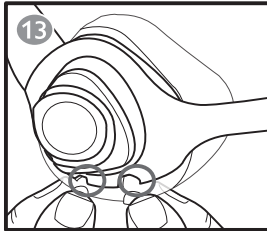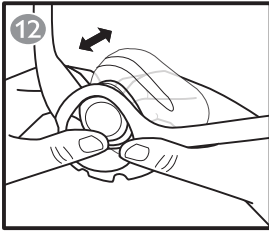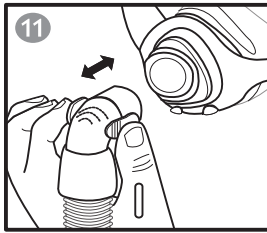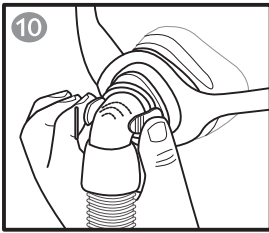

Disassembly / Assembly

Reversible Fabric Frame

Choose the fabric that feels comfortable against the skin. That side will face inward toward the cushion and headgear. The clear frame is not reversible.

Removing the Elbow and Tubing

- Squeeze the Quick Release Tabs (10).
- Pull the elbow away from the cushion (11).

Removing the Cushion

Push the cushion backwards through the frame (12).

Removing the Headgear

- Detach the headgear bottom clips from the frame or undo the bottom tabs and pull.
- Undo the headgear top tabs and pull the straps through the frame.

Attaching the Cushion

1. Push the cushion into the frame towards the front and press into place (12).
2. The bottom of the cushion will match the alignment cutouts of mask frame (13).

Attaching the Elbow and Tubing




Push the elbow into the cushion until the Quick Release Tabs click into place (11).

Attaching the Headgear

1. Hold the crown strap upward.
2. Make sure the wide strap marked with "Philips Respirationics" is outward.
3. Slide the headgear tabs into the longer frame arm slots.
4. Connect the headgear clips to the frame or slide the tabs into the shorter frame arm slots if not using the headgear clips.

Optional: Disassembly/Assembly of Magnetic Headgear Clips

If your Wisp mask contains magnetic headgear clips (14), the following instructions apply: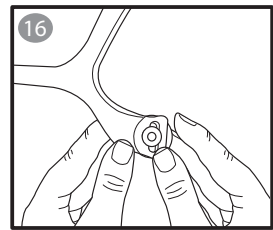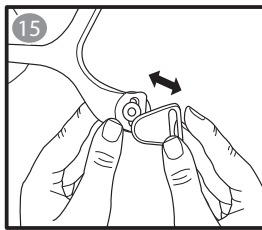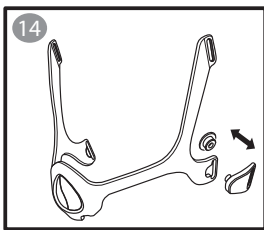

Removing the Magnetic Headgear Clips

1. Pull the front end of the magnetic clip away from the back circular end of the clip to disconnect the magnets. (15)
2. Push the circular end back and out of the mask frame bottom slot. (16) You may need to flex the mask frame for easier removal.

Attaching the Magnetic Headgear Clips

1. Push the back circular end of the magnetic headgear clip into the mask frame bottom slot until it clicks into place. (16)
2. Connect the front end of the magnetic clip to the back circular end. The clip will snap into place when the magnetic ends are connected. (15)

Philips Respirationics System One Resistance Control

This mask when combined with a Philips Respirationics System One device, provides optimal resistance compensation. This mask value is X1 and should be set by the provider.


ⓘ Notes:

- Compare the mask to the device. See the device manual if the values do not match.
- System One is not compatible with masks that use a separate exhalation device.

Specifications

⚠ Warning: The technical specifications of the mask are provided for the healthcare professional to determine if it is compatible with the CPAP or bi-level therapy device. If used outside these specifications, or if used with incompatible devices, the mask may be uncomfortable, the seal of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect device function. The pressure flow curve shown below is an approximation of expected performance. Exact measurements may vary.

Pressure Flow Curve

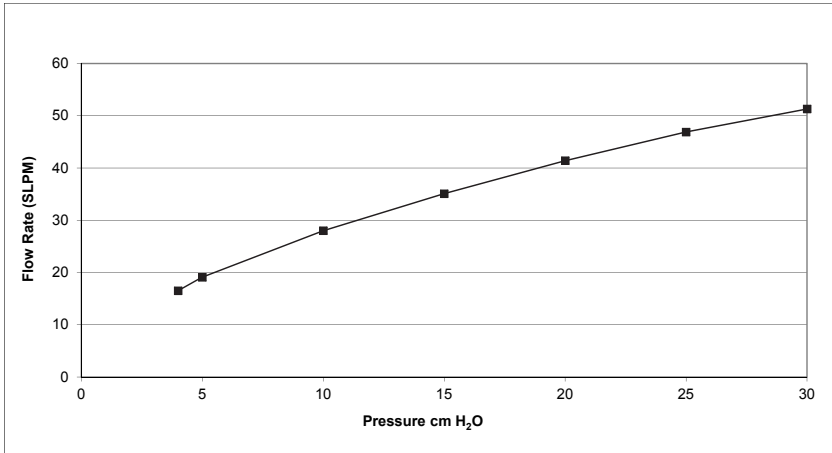Resistance

Drop in Pressure at

	50 SLPM	100 SLPM
All sizes	0.8 cm H ₂ O	3.2 cm H ₂ O

Deadspace Volume

P	37 mL
S/M	49 mL
L	73 mL
XL (adult only)	85 mL

Sound Levels

A-weighted Sound Power Level	27 dBA
A-weighted Sound Pressure Level at 1 m	19 dBA

Disposal

Dispose of in accordance with local regulations.

Storage Conditions

Temperature: -4° to 140° F (-20° to +60° C)
Relative Humidity: 15% to 95% non-condensing

Limited Warranty

Respironics, Inc. warrants that its mask systems (including mask frame and cushion) (the "Product") shall be free from defects of workmanship and materials for a period of ninety (90) days from the date of purchase (the "Warranty Period").

If the Product fails under normal conditions of use during the Warranty Period and the Product is returned to Respironics within the Warranty Period, Respironics will replace the Product. This warranty is nontransferable and only applies to the original owner of the Product. The foregoing replacement remedy will be the sole remedy for breach of the foregoing warranty.

This warranty does not cover damage caused by accident, misuse, abuse, negligence, alteration, failure to use or maintain the Product under conditions of normal use and in accordance with the terms of the product literature, and other defects not related to materials or workmanship.

This warranty does not apply to any Product that may have been repaired or altered by anyone other than Respironics. Respironics disclaims all liability for economic loss, loss of profits, overhead, or indirect, consequential, special or incidental damages which may be claimed to arise from any sale or use of the Product. Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

THIS WARRANTY IS GIVEN IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IN ADDITION, ANY IMPLIED WARRANTY, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IS SPECIFICALLY DISCLAIMED. SOME JURISDICTIONS DO NOT ALLOW DISCLAIMERS OF IMPLIED WARRANTIES, SO THE ABOVE LIMITATION MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS UNDER LAWS APPLICABLE IN YOUR SPECIFIC JURISDICTION.

To exercise your rights under this limited warranty, contact your local authorized Respironics, Inc. dealer or Respironics, Inc. at 1001 Murry Ridge Lane, Murrysville, Pennsylvania 15668, USA, or Respironics Deutschland GmbH & Co KG, Gewerbestraße 17, 82211 Herrsching, Germany.

Limited Warranty for Australia and New Zealand Only

Note: For Australian and New Zealand customers this warranty replaces the warranty contained above.

Respironics, Inc., a Philips Healthcare company warrants that the Products shall be free from defects of workmanship and materials and will perform in accordance with the Product specifications. 2. This warranty is valid for a period of ninety (90) days from the date of purchase from an authorised Respironics dealer. 3 If the Product is found to contain a defect of workmanship or materials or fails to perform in accordance with the Product specifications during the applicable warranty period, Respironics will repair or replace, at its option, the defective material or part. 4. The customer is responsible for returning the product to an authorised Philips Respironics dealer, and collecting the product from the authorised Philips Respironics dealer after repair or replacement, at its own cost. Philips Respironics is responsible only for the freight cost of transporting the product between the authorised Philips Respironics dealer and Respironics. Respironics reserves the right to charge an evaluation and postage fee for any returned Product as to which no problem is found following investigation. 5. This warranty does not cover damage caused by accident, misuse, abuse, alteration, and other defects not related to materials or workmanship. 6. The warranty provided by Respironics herein is not transferrable by the Buyer in the event of any sale or transfer of Products purchased by the Buyer from an authorised Respironics dealer. 7 To the extent permitted by law, where the Buyer has the benefit of an implied guarantee under the Australian Consumer Law, but the Product is not of a kind ordinarily acquired for personal, domestic or household use or consumption Respironics' liability shall be limited, at the option of Respironics, to the replacement or repair of the Product or the supply of an equivalent Product. 8. To exercise your rights under this warranty, contact your local authorised Philips Respironics dealer. A list of all authorised dealers is available at the following link: http://www.healthcare.philips.com/au_en/homehealth/distributors_index.wpd.

Alternatively, you can make a claim under this warranty by contacting Respironics directly at: Philips Electronics Australia Limited, 65 Epping Road, North Ryde NSW 2113, Australia. Tel: 1300 766 488, Email: prcontact@philips.com. 9. The following statement is provided to a Buyer who is a "consumer" under the Australian Consumer Law: Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the good repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount

to a major failure. 10. The following statement is provided to a Buyer who is a “consumer” under the Consumer Guarantees Act 1993, New Zealand: Our goods come with guarantees that cannot be excluded under the Consumer Guarantees Act 1993. This guarantee applies in addition to the conditions and guarantees implied by that legislation.

AUSTRALIAN SPONSOR DETAILS:

Philips Electronics Australia Ltd.
65 Epping Road, North Ryde, NSW 2113
Australia

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© 2017 Koninklijke Philips N.V. All rights reserved.