Manufacturer
WEINMANN Emergency
Medical Technology GmbH + Co. KG
Frohbösestraße 12
22525 Hamburg
GERMANY
T: +49 40 88 18 96-120
E: customerservice@weinmann-emt.de

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Description

Intended use

The ventilation masks are used for emergency medical purposes.

In cases of manual or automatic ventilation the masks create an airtight connection between the patient and the ventilation appliance or ventilation aid.

Function

The mask is placed over the patient's nose and mouth region and pressed against the face. The ventilation gas passes through the mask to the patient. The shape and the flexible material of the edges of the mask ensure a seal between the mask and the face.

Safety

For your own safety, the safety of your patients, and to comply with the requirements of EU Directive 93/42/EEC, please observe the following points:

- Please read these operating instructions carefully. They belong to the mask and must be kept available at all times.
- If third-party items are used, functional failures may occur and fitness for use may be restricted. Biocompatibility requirements may also not be met. Please note that in such cases, any claim under warranty and liability will be voided if neither the accessories nor genuine replacement parts recommended in the instructions for use are used.
- Select a ventilation pressure that is suitable for the patient and the medical indication. Note that the pressure must **on no account** be greater than 60 mbar. Excessive pressure may result in over-expansion of the stomach!
- If you use an HME filter, this automatically increases the deadspace. In the case of small ventilation volumes, e.g. babies and small children, this may result in the patient receiving insufficient supplies of the ventilation gas.
- Please follow the instructions in the section entitled “Hygienic preparation” to avoid infection or bacterial contamination.

Operation

Preparing for ventilation

1. Connect the cone **1** of the mask to the valve of the ventilation appliance or ventilation aid.

In order to humidify the ventilation gas you can insert an HME filter between the mask and the ventilation appliance or ventilation aid.

2. Press the mask over nose and mouth and start ventilation.

After ventilation

1. Remove the mask from the ventilation appliance or ventilation aid.
2. Clean/disinfect the mask in accordance with the section entitled “Hygienic preparation”.

Pressure equalization in rim of mask

The mask rim **2** of the clear masks is pressureless. Under certain conditions, e.g. if the mask has been lying in the sun for a long time, it may happen that over-pressure or under-pressure is created in the rim, and this then needs to be equalized:

1. Pull the plug **3** out of the extension.

1. Wait until the pressure has equalized.
2. Replace plug **3** firmly.

Hygienic preparation

Be sure to clean or disinfect the mask after every use.

The mask must be disinfected or sterilized before being used by a different patient.

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- EN**
- Clean the mask with hot water and a household cleaner. You can also clean the mask in a dishwasher at up to 100 °C.
 - To disinfect the mask, use a dilute solution of disinfectant. Please observe the instructions for use supplied with your disinfectant. We recommend GIGASEPT FF. You are recommended to wear suitable gloves (e.g. household or disposable gloves) during disinfection procedures.
 - Sterilization is possible at a temperature of up to 134 °C.
 - Notes on the hygiene treatment and on the hygiene treatment for a change of patient can be found in a brochure from our information portal at www.weinmann-emergency.com. If you do not have Internet access, contact WEINMANN Emergency.

Maintenance

The ventilation masks are maintenance free.

Functional Check

Please carry out a visual inspection before every use.

If the mask displays cracks or tears, you can no longer be sure that it is free from leaks, and it must therefore be disposed of. Dispose of used masks as domestic garbage.

Technical Data

Product category according to 93/42/EEC	Ila
Connecting cone (according to EN ISO 5356-1)	22 mm internal; 15 mm external, for clear mask size 1,
Temperature range	
– Operation	-20 °C to +60 °C
– Storage	-40 °C to +70 °C
Material	
– Rendell-Baker masks	Silicone
– Mask rim	Silicone
– Attachment ring	Silicone
– Mask body	Polysulfone

Warranty

WEINMANN Emergency gives the customer a limited manufacturer warranty on new genuine WEINMANN Emergency products and any replacement part fitted by WEINMANN Emergency in accordance with the warranty conditions applicable to the product in question and in accordance with the warranty periods from date of purchase as listed below. The warranty conditions can be downloaded from www.weinmann-emergency.com on the Internet. We can also send you the warranty conditions on request. In the event of a claim under warranty, contact your specialist dealer.

Product	Warranty period
Masks including accessories, rechargeable battery, batteries (unless there is information to the contrary in the technical documentation), sensors, tube systems	6 months
Disposable products	None

CE 0197 Subject to technical change without notice

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