Manufactured for:
Respironics Inc.
1001 Murry Ridge Lane
Murrysville, PA 15668 USA

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ENGLISH

Intended Use

The FitLife SE Total Face Mask is intended to provide a patient interface for application of noninvasive ventilation. The mask is to be used as an accessory to ventilators which have adequate alarms and safety systems for ventilation failure, and which are intended to administer CPAP or positive pressure ventilation for treatment of respiratory failure, respiratory insufficiency or obstructive sleep apnea. This mask is for single patient use in the home or multi-patient use in the hospital/institutional environment only. The small size mask is to be used on patients 7 years or older (>20 kg) who are appropriate candidates for noninvasive ventilation. The large and extra large size masks are to be used on patients (>30 kg) who are appropriate candidates for noninvasive ventilation.

 **Note:** An exhalation port is not built into the FitLife SE Total Face Mask. A separate exhalation device must be used with this mask.

 **Note:** This mask is not made with natural rubber latex or DEHP.

Symbols

Warning or
Caution

Note

Tip

Consult instructions
for use

Not made with
natural rubber latex

Warnings:

- This mask is not suitable for providing life support ventilation.
- This mask requires a separate exhalation device.
- This mask is designed for use with CPAP or bi-level systems recommended by your health care professional or respiratory therapist. Do not wear this mask unless the CPAP or bi-level system is turned on and operating properly. **Do not block or try to seal the exhalation port. Explanation of the Warning:** CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. This warning applies to most models of CPAP systems.
- If oxygen is used with the device, the oxygen flow must be turned off when the device is not operating. **Explanation of the Warning:** When the device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the device enclosure. Oxygen accumulated in the device enclosure will create a risk of fire.
- Oxygen supports combustion. Oxygen should not be used while smoking or in the presence of an open flame.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP and bi-level machines.
- Some users may experience skin redness, irritation, or discomfort. If this happens, discontinue use and contact your healthcare professional.
- The patient's physician should be contacted if the patient experiences the following symptoms while using the mask or after removing it: Unusual chest discomfort, shortness of breath, stomach distension, belching, or severe headache; drying of the eyes, eye pain, or eye infections; blurred vision. (Consult an ophthalmologist if symptoms persist.)
- At low CPAP or EPAP pressures, the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.

- This mask should not be used for patients who are uncooperative, obtunded, unresponsive, or unable to remove the mask by themselves.
- This mask is not recommended for patients who are taking a prescription drug that may cause vomiting.
- Appropriate patient monitoring should be used as medically necessary.
- The mask contains small parts which could result in a choking hazard.
- If an additional exhalation device is added to the patient circuit, you may need to adjust the pressure level to compensate for the additional leak of the exhalation device.
- This mask does not include an anti-asphyxia valve that would allow a patient to breathe if the ventilator failed.
- Hand wash prior to first use. Inspect the mask for damage or wear (cracking, crazing, tears, etc). Discard and replace any components as necessary.
- Use of a nasal or full face mask may cause tooth, gum, or jaw soreness or aggravate an existing dental condition. Consult your physician or dentist if symptoms occur.

Contraindications

This mask may not be suitable for persons with the following conditions: glaucoma, recent eye surgery or dry eyes, impaired cardiac sphincter function, excessive reflux, impaired cough reflex, and hiatal hernia. It should not be used if the patient is unable to remove the mask by themselves.

Before Use

- Read and understand the instructions completely.
- Hand wash the mask.
- Wash your face.
- Verify that the therapy device, i.e., ventilator, including the alarms and safety systems, has been validated prior to use.
- Check for the blue elbow on the mask, and note that the mask does not include an anti-asphyxia valve that would allow the patient to breathe if the ventilator failed.
- If a nasogastric (NG) tube or similar device is in place, use the optional NG tube sealing pad. Position the pad so that its flat surface is against the patient's face and the C-shaped opening surrounds the tube.
- Inspect the mask components for damage or wear; replace if necessary.
- Verify therapy device pressure(s).

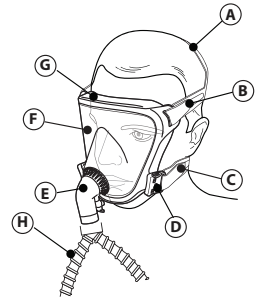
Figure 1

- A = Top Strap
- B = Side Strap
- C = Bottom Strap
- D = Headgear Clip
- E = Blue Standard Elbow (SE)
- F = Faceplate
- G = Cushion
- H = Dual Limb Tubing (not included)

Cleaning Instructions

Hand wash the mask before first use and daily. The headgear should be hand washed weekly, or as needed. The headgear does not need to be removed for daily cleaning of the mask.

1. Hand wash mask in warm water with a mild liquid dishwashing detergent.

⚠Caution: Do not use bleach, alcohol, cleaning solutions containing bleach or alcohol, or cleaning solutions containing conditioners or moisturizers.

⚠Caution: Any deviation from these instructions may impact the performance of the product.

2. Rinse thoroughly. Air dry completely before use. Make sure the mask is dry before use. Lay the headgear flat or line dry. Do not place the headgear into the dryer.

⚠Caution: Inspect the mask for damage or wear (cracking, crazing, tears, etc). Discard and replace any components as necessary.

Institutional Disinfection

For multi-patient use in the hospital/institutional environment, use the Disinfection Guide to reprocess the mask between patients. These instructions can be obtained by visiting us online at www.philips.com/respironics or by contacting Philips Respironics Customer Service at 1-724-387-4000 or Respironics Deutschland at +49 8152 93060.

Dishwasher Instructions

⚠ Caution: Use a mild liquid dishwashing detergent only.

1. Remove the headgear. Do not wash the headgear in the dishwasher.
2. Wash in the top shelf of the dishwasher.
3. Air dry. Make sure the mask is dry before use.
4. Inspect all parts for damage or wear; replace any parts that have visibly deteriorated (cracking, crazing, tears, etc). Replace the mask if the cushion becomes hardened.

Putting on the Mask

1. Loosen the side headgear straps by peeling back the tabs and open the straps to a larger size.
2. Disconnect one or both of the bottom headgear clips. Grasp the clip with the thumb and index finger and in a turning motion, lift the clip upwards with the thumb while pulling the clip away from the mask faceplate.
3. Hold the mask against your face. The top of the cushion will contact your face just above the eyebrows. The bottom of the cushion will contact your face just above the chin. Be sure your mouth remains slightly open.
4. Pull the headgear over your head.
5. Attach one or both bottom headgear clips by finding the tab on the lower part of the mask faceplate with your thumb (Figure 2). Guide and gently push the headgear clip onto the tab with your middle and index fingers until it clicks into place.

Figure 2

Achieving the Right Fit

1. Tighten the side headgear straps. The top strap should lie at the crown of the head. The cross-strap should sit low on the back of the head.
2. Tighten the bottom headgear straps.
3. Adjust the top headgear strap, if necessary. If the side headgear straps are resting on your ears, adjust the top headgear strap to lift the straps slightly off of your ears. DO NOT over tighten as it may cause the headgear to slip.
4. Connect the flexible dual-limb tubing from the non-invasive ventilator or therapy device to the 22 mm blue standard elbow (SE).
5. With the airflow on, lie down and breathe normally with your mouth slightly open. Re-seat the mask by pulling it directly away from your face and gently setting it back into place.
6. Assume different sleeping positions and move your head around.
7. If there are any air leaks, make any final adjustments. Some air leakage is normal.

Comfort Tips

- Re-seat the mask. Pull the mask by the sides of the faceplate directly away from the face then gently set it back into place. DO NOT pull the mask by the standard elbow. This allows the cushion to create a new seal on the face.
- Adjust the side headgear straps to eliminate leaks around the forehead and temples.
- Adjust the bottom headgear straps to eliminate leaks around the cheeks and chin.
- The most common mistake is over tightening the headgear. The headgear should fit loose and comfortable. If your skin bulges around the mask or if you see red marks on your face, loosen the headgear.

Figure 3

Removing the Mask

Undo one or both bottom headgear clips and pull over the top of your head (Figure 3). By undoing the headgear clips, instead of the tabs, you will preserve your adjustments.

Replacing the Standard Elbow

To remove: Hold the faceplate swivel and gently pull the elbow from the faceplate swivel (Figure 4).

To replace: Press the standard elbow into the faceplate swivel. Do not use excessive force or any tools.



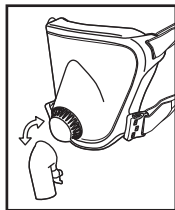
Figure 4

Specifications

Warning: The technical specifications of the mask are provided for your healthcare professional to determine if it is compatible with your CPAP or bi-level therapy device. If used outside these specifications, or if used with incompatible devices, the mask may be uncomfortable, the seal of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect device function.

Pressure Flow Curve

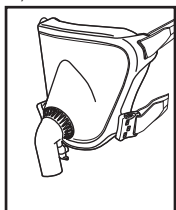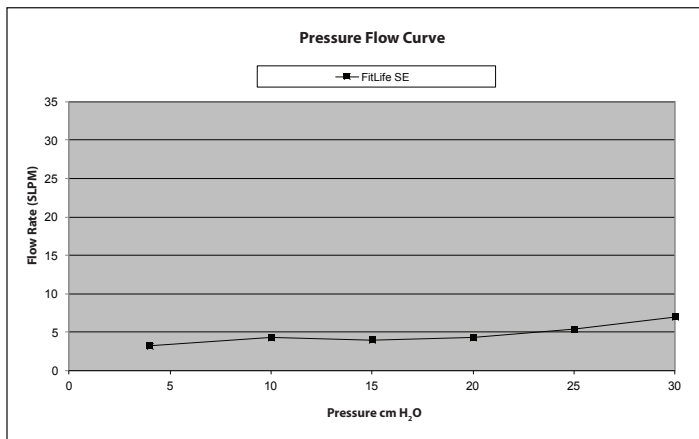Resistance

Drop in Pressure at

	50 SLPM	100 SLPM
Small/Large/Extra Large	0.08 cm H ₂ O	0.8 cm H ₂ O

Deadspace Volume

Small = 375 mL

Large = 550 mL

Extra Large = 717 mL

Sound Levels

A-weighted Sound Power Level ≤ 22 dBA

A-weighted Sound Pressure Level at 1 m ≤ 14 dBA

Disposal

Dispose of in accordance with local regulations.

Storage Conditions

Temperature: -20° to $+60^{\circ}$ C

Relative Humidity: 15% to 95% non-condensing

Limited Warranty

Respironics, Inc. warrants that its mask systems (including mask frame and cushion) (the "Product") shall be free from defects of workmanship and materials for a period of ninety (90) days from the date of purchase (the "Warranty Period").

If the Product fails under normal conditions of use during the Warranty Period and the Product is returned to Respironics within the Warranty Period, Respironics will replace the Product. This warranty is nontransferable and only applies to the original owner of the Product. The foregoing replacement remedy will be the sole remedy for breach of the foregoing warranty.

This warranty does not cover damage caused by accident, misuse, abuse, negligence, alteration, failure to use or maintain the Product under conditions of normal use and in accordance with the terms of the product literature, and other defects not related to materials or workmanship.

This warranty does not apply to any Product that may have been repaired or altered by anyone other than Respironics. Respironics disclaims all liability for economic loss, loss of profits, overhead, or indirect, consequential, special or incidental damages which may be claimed to arise from any sale or use of the Product. Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

THIS WARRANTY IS GIVEN IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IN ADDITION, ANY IMPLIED WARRANTY, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IS SPECIFICALLY DISCLAIMED. SOME JURISDICTIONS DO NOT ALLOW DISCLAIMERS OF IMPLIED WARRANTIES, SO THE ABOVE LIMITATION MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS UNDER LAWS APPLICABLE IN YOUR SPECIFIC JURISDICTION.

To exercise your rights under this limited warranty, contact your local authorized Respironics, Inc. dealer or Respironics, Inc. at 1001 Murry Ridge Lane, Murrysville, Pennsylvania 15668, USA, or Respironics Deutschland GmbH & Co KG, Gewerbestraße 17, 82211 Herrsching, Germany.

AUSTRALIAN SPONSOR DETAILS:

Philips Electronics Australia Ltd.

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© 2015 Koninklijke Philips N.V. All rights reserved.